

COVID-19

Information leaflet

Grade III Social Mask re-usable fabric



These textile masks are for general public use in the context of their professional and authorized social activities, including inside indoor spaces.

Important:

Grade II and III masks are not considered Personal Protection Equipment or suitable to be worn in a clinical context.

Manufacturer: Pedrosa & Rodrigues, S.A.

Address: Rua dos Escuteiros 42, Gilmonde, 4755-230 Barcelos, Portugal


Article reference: 200518

Batch number | Month | Year: 850900 | April | 2020 (for certification purposes)


Fabric composition: 100% Cotton

Size range: Adult, one size

Front

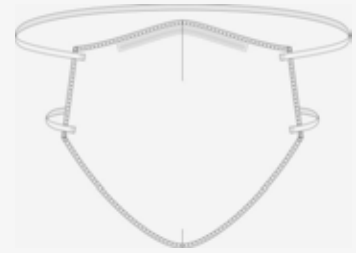


Side

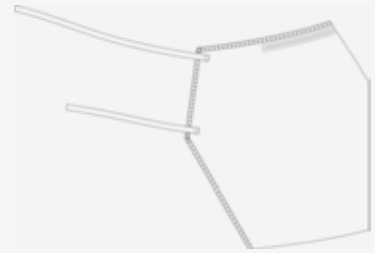


Instructions:

- This mask is a complement to health and safety guidelines and social distancing recommendations in response to COVID-19. Further information at: <https://www.who.int/emergencies/diseases/novel-coronavirus-2019>
- This re-usable mask must be handled as per World Health Organization guidelines:
- Hands must be washed with water and soap or an alcohol-based gel before this mask is applied;
- Place the mask upon the nose and mouth area, with the inside towards the face and outside facing out;
- Adjust the metal strip over the nose and ensure the nose, mouth and chin are fully covered without any gaps;
- Do not touch the mask while worn. If this happens, hands must be washed immediately;
- Recommended wear is 4 hours maximum. Should it become humid or after this 4 hour period, the mask should be replaced and washed before being used again;
- If this mask gets wet it must be replaced by a new one as soon as possible. The wet mask must be washed before being used again;
- To remove the mask, please handle the elastic hoops behind the ears only. Avoid touching the front of the mask at all times;
- Do not keep the mask around the neck in between wears;
- If the mask cannot be washed right after use, it is recommended that it is kept inside a closed plastic bag;
- Hands must be washed after handling a mask that has been worn;
- The mask must be washed after each wear, and the manufacturer's certified number of washes complied with;
- The mask offers guaranteed protection if these instructions are followed;
- The mask must be kept in a dry environment and away from potential contamination both prior to and after washing;
- In case of defect, do not wear this mask and replace it with a new one;
- At the end of the mask's lifecycle, the mas must be washed and then discarded as non-hazardous textile waste. If discarded before wash, it must be treated as bio-hazard.

How to apply the mask:

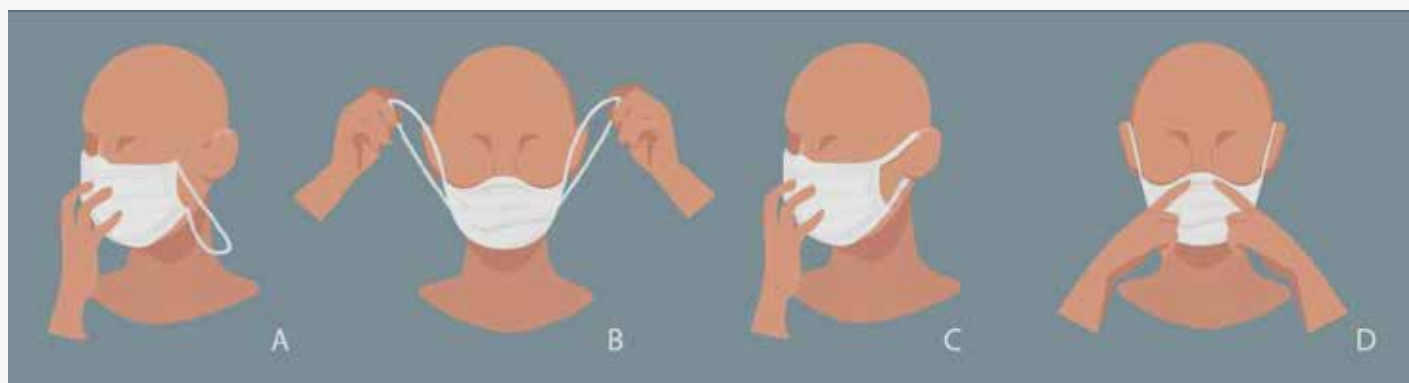

How to remove the mask:

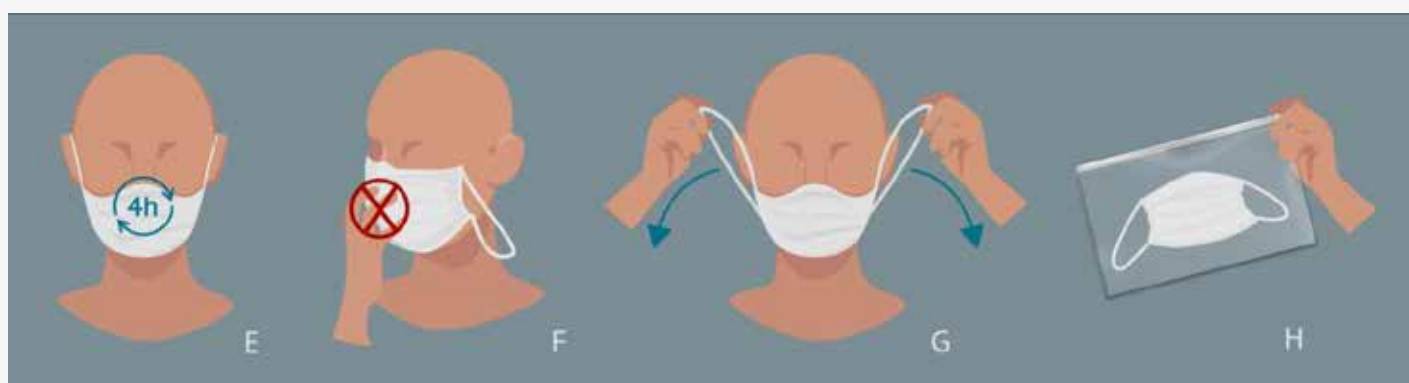

Care instructions:

Number of washes with guarantee of protection: 15

Masks must be washed in a normal, full cycle at 60°C with detergent, after each wear. Please ensure that the mask is completely dry before wearing it again.



60°C washing allowed



Do not dry clean

Compliance verified via report number "CITEVE n.° 000/2020".